


For Active Days, Give your Pod a Pal.

Looking for additional support during the warmer months, sports or water activities? We've got you covered. Try PodPals™, an adhesive overlay accessory for the Pod developed by the makers of the Omnipod® Insulin Management System!

FLEXIBLE DESIGN

MEDICAL GRADE

CUSTOM-FIT FOR THE POD

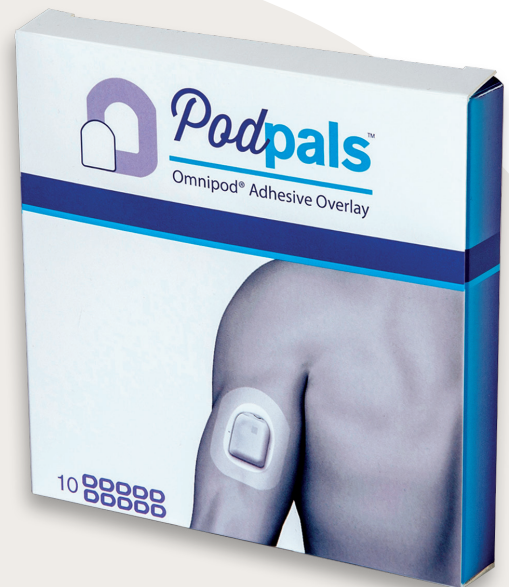


**TO ORDER: JUST CALL INSULET CUSTOMER CARE TEAM :
[0800 011 6132]† OR [+44 20 3887 1709†] WHEN CALLING
FROM ABROAD. AVAILABLE 24 HOURS A DAY, 7 DAYS A WEEK.**




Podpals™ instructions for use


PodPals™ are intended to be accessories that may be used to assist in adherence of the skin attachment area surrounding the Pod while using the Omnipod® System or Omnipod DASH® System.

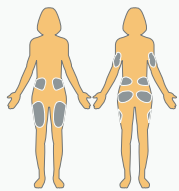


1 > Begin with an active Pod in an appropriate site location.

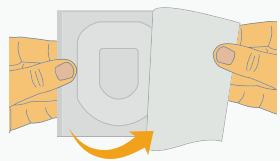


2 > Open the package by pulling the tabs apart.

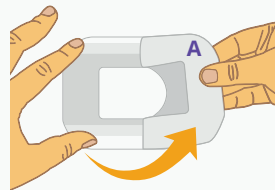


3 > Peel back and completely remove the top layer of paper backing (A).

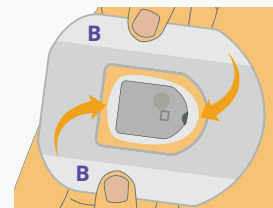


4 > With the sticky side facing down, fit the PodPal™ over the Pod. Avoid touching the adhesive. Instead, hold using the paper backings (B). Do not stretch the PodPal™ prior to, or during application.

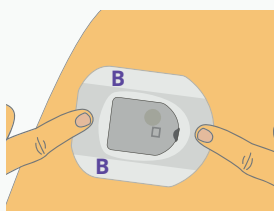


5 > Before pressing down to secure the PodPal™, make sure it covers all edges of the Pod's adhesive pad, and does not cover any part of the Pod, especially the cannula viewing window.

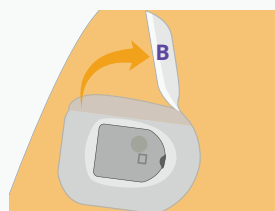

6 > Gently peel off the paper backings (B) from both sides.

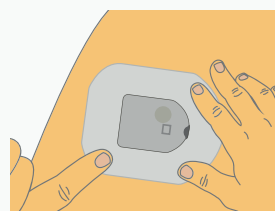

7 > Press down on all areas to secure the PodPal™ to the skin and the Pod's adhesive pad. Make sure to smooth out any wrinkles.

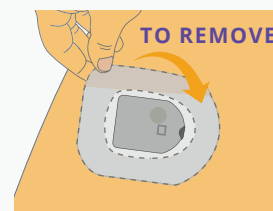

8 > When ready to remove your PodPal™, gently peel up on its edges and lift from the skin. Remove the PodPal™ first, then the Pod.

APPLICATION TIPS

- Make sure the application site is clean and thoroughly dry before application. Avoid using body lotions, creams or oils, as these products may loosen the adhesive.
- Press down and hold the PodPal™ for 30 seconds after application to ensure maximum adhesion.
- Press down and smooth out the overlay as needed to avoid wrinkles.
- When applying your PodPal™ in a hard-to-reach site location, assistance may be required.
- Once applied, the PodPal™ must remain in place for the life of the Pod.

