



# Omnipod® 5 & Glooko® Key information

AN OMNIPOD® 5 LEAFLET FOR  
PEOPLE LIVING WITH DIABETES



## LINK your Omnipod 5 Automated Insulin Delivery System data with Glooko

If you have missed linking your Omnipod 5 and Glooko, please follow these four simple steps:

1. Log into your online Omnipod account at [www.omnipod.com](http://www.omnipod.com), using your Omnipod ID
2. Go to the **My Account** tab and then select Account linking from the drop down menu. Then click “**Link**”
3. Select “**sign up**” or “**log in**” with Glooko
4. Proceed to Glooko and complete the process to link your account

When complete, your status will update to “**Linked**” and data will automatically upload into your personal Glooko account\*. You can always unlink your account between Omnipod and Glooko to stop data sharing.



NOTE: All screen images are examples, for instructional purposes only.

\*Must be connected to WiFi or cellular data, and Omnipod 5 users must have WiFi or cellular data access to receive updates in Glooko.

Choose to **SHARE** your  
Omnipod 5 Automated  
Insulin Delivery  
System data with your  
healthcare provider



*Entering a ProConnect code in  
Glooko is the only way for your  
healthcare provider to see your  
Omnipod 5 data on their computer.*

**ProConnect Code:  
Provided by your  
healthcare practitioner**

If you have missed linking  
your Glooko account to your  
Healthcare Professional, please  
follow these five simple steps:

1. Log into your Glooko account at [my.glooko.com](https://my.glooko.com)
2. Select “**Settings**” from the drop-down menu  
(top right on the home screen)
3. Scroll down to Account and select  
“**+ Add New Code**”
4. Select “**Continue**”
5. Enter ProConnect Code (listed below)  
then select “**Submit**”

NOTE: All screen images are examples, for instructional purposes only.

## Using the GLOOKO MOBILE APP

*You have the choice to use the Glooko mobile app or the online Glooko platform [my.glooko.com](https://my.glooko.com)*



NOTE: All screen images are examples, for instructional purposes only.

For iPhone



### Download the app and login

- From the App Store search for and install the “Glooko mobile app”
- Log in or create an account by filling out the requested information


### View your data

Once your data is uploaded to Glooko, you can view your trends and graphs on your iPhone.

### Choose to share data with your healthcare provider

- Click the **More** tab,
- Click **Share Reports** to email or fax the reports

## For Android

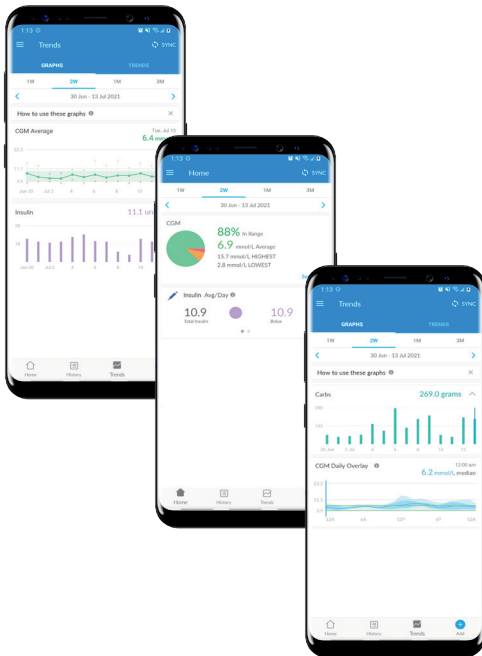

### Download the app and login

- From the Google Play™ Store search for and install the “Glooko mobile app”
- Log in or create an account by filling out the requested information

### Sync and view your data

- Tap Sync from the Home Screen, and select Omnipod 5 Automated Insulin Delivery System
- Follow the prompts to complete the sync

### Choose to share data with your healthcare provider

Click **Share Reports** to email or fax the reports.

NOTE: All screen images are examples, for instructional purposes only.

## Create, Store or Print a **PDF REPORT**

You can create and view reports on the **Glooko mobile app** or at **my.glooko.com**.

**1. Login** to your account

**2. To create a report:**

- Click on your **Profile**
- Click **Create PDF Report** (top right)
- Choose desired reports and time frame and click **Create PDF**

**3. To save/print a report:**

Once the report is created, either save it to your desktop or a desired location, or print directly.

Create PDF

Liv Waddle  
DOB: 17/08/09  
Diabetes: Type 1

Time: 2 weeks 10/11/2021 - 23/11/2021

Preferred PDF Setting: No Profile Available

Summary Logbook Overview Daily Overview Week View

Overlay 2 pages Calendar 2 pages Insights 1 page Devices 14 pages

Estimated report length: 20

Print mode:  Black and White  Colour

Type comment here! Comments will appear in the Summary report section.

Save selection as Favourite Profile **Create PDF**

NOTE: All screen images are examples, for instructional purposes only.

**TIP! Save Customized Report Sets:**

- Click **Settings**
- Scroll down and click **New Favourite**
- Select desired reports, and name the report set (*i.e. Omnipod+Dexcom reports or Dr. Smith reports*)



# Great reports to get started with are:


## Summary reports, Daily, Weekly

Snapshot of **key statistics** and **aggregated information** to identify positives and challenges to support patient care



## The Device Settings Report

When you upload your data to Insulet Provided Glooko, your pump settings will be saved. This report is a great resource for you and your care team to reference.

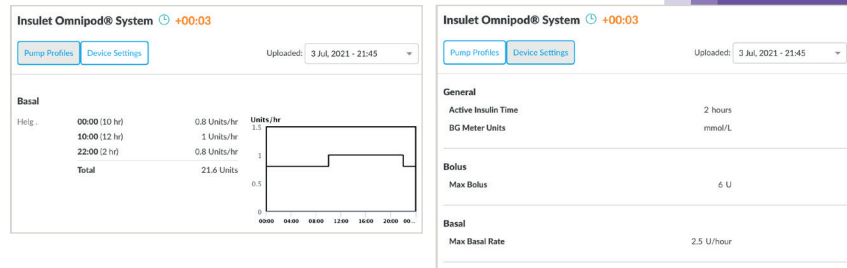

NOTE: All screen images are examples, for instructional purposes only.



If you have any questions or concerns regarding Omnipod 5 please contact the Omnipod team on:

**0800 011 6132\***

or +44 20 3887 1709 if calling from abroad.

If you are experiencing any issues with Glooko, please contact Glooko Support.

**020 7795 8191\***

[www.glooko.com](http://www.glooko.com)  
[uk@glooko.com](mailto:uk@glooko.com)



\*Your call be monitored and recorded for quality monitoring purposes. Calls to 0800 numbers are free from local landlines, but other networks may charge for these calls.

©2023 Insulet Corporation. Omnipod, the Omnipod logo, DASH, the DASH logo, SmartAdjust, Simplify Life and Podder are trademarks or registered trademarks of Insulet Corporation in the United States of America and other various jurisdictions. Dexcom and Dexcom G6 are registered trademarks of Dexcom, Inc. and used with permission. Glooko is a trademark of Glooko, Inc. and used with permission. All rights reserved. Insulet Netherlands BV Stadsplateau 7, 3521 AZ Utrecht, Netherlands. INS-OHS-03-2023-00048v1